MASSAGELIEGE

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Description of Symbols

NOTE
Indicates important information and instructions throughout the manual.

CAUTION
Contains important information on how to avoid damage to the equipment.

DANGER
Contains important information on how to avoid personal injuries.

Operating and safety instructions

We kindly ask you to follow the following instructions to avoid damage to persons and/or equipment:

- Read the instruction manual carefully before using the product.
- Improper use may cause damage to persons and/or equipment.
- The maximum user weight is 125 kg. The maximum work load of the massage table is 225 kg. Do not apply more than the maximum load on the massage table (follow the “Instructions on work load” on page 8).
- Before each use, check for faults such as damage or loose screw connections. Also ensure that the ropes are not worn and that the table is fully extended. If damage occurs, do not use the equipment until it is repaired.
- Ensure that all four legs have the same height and stand on a solid surface when the equipment is used.
- Please note that the surface temperature may be increased by direct sunlight. Due to that fact, do not unnecessarily expose the equipment to direct sunlight.
- Do not position the equipment on or close to easily inflammable material. Keep away from fire, cigarettes and great heat. The material is not flame-retardant.
- The equipment is in no way suitable as a toy. Children should not be allowed near the equipment without supervision. Due to their natural play instinct, they might use it in a manner which is not intended.
- The owner has the responsibility of providing adequate information on possible dangers to all other persons using the equipment.
- Do not perform unauthorised alterations on the product that have not been explicitly authorised by our service personnel. This not only terminates your warranty but we also assume no liability for possible negative consequences.
- Never adjust the position of a moveable part with a tool. Please note that, as a rule, moveable parts always pose a risk of crushing body parts.
- Keep plastic bags from the packaging away from children.
- Please store this instruction manual for the entire time of use.

The massage table is for private and home use only and serves as reclining area for people.

Packaging


Environmentally friendly, reusable materials:

- Outer packaging made of cardboard
- Films and bags made of polyethylene (PE)

Disposal

Please ensure an environmentally friendly disposal in compliance with local regulations!

Characteristics of the massage table features

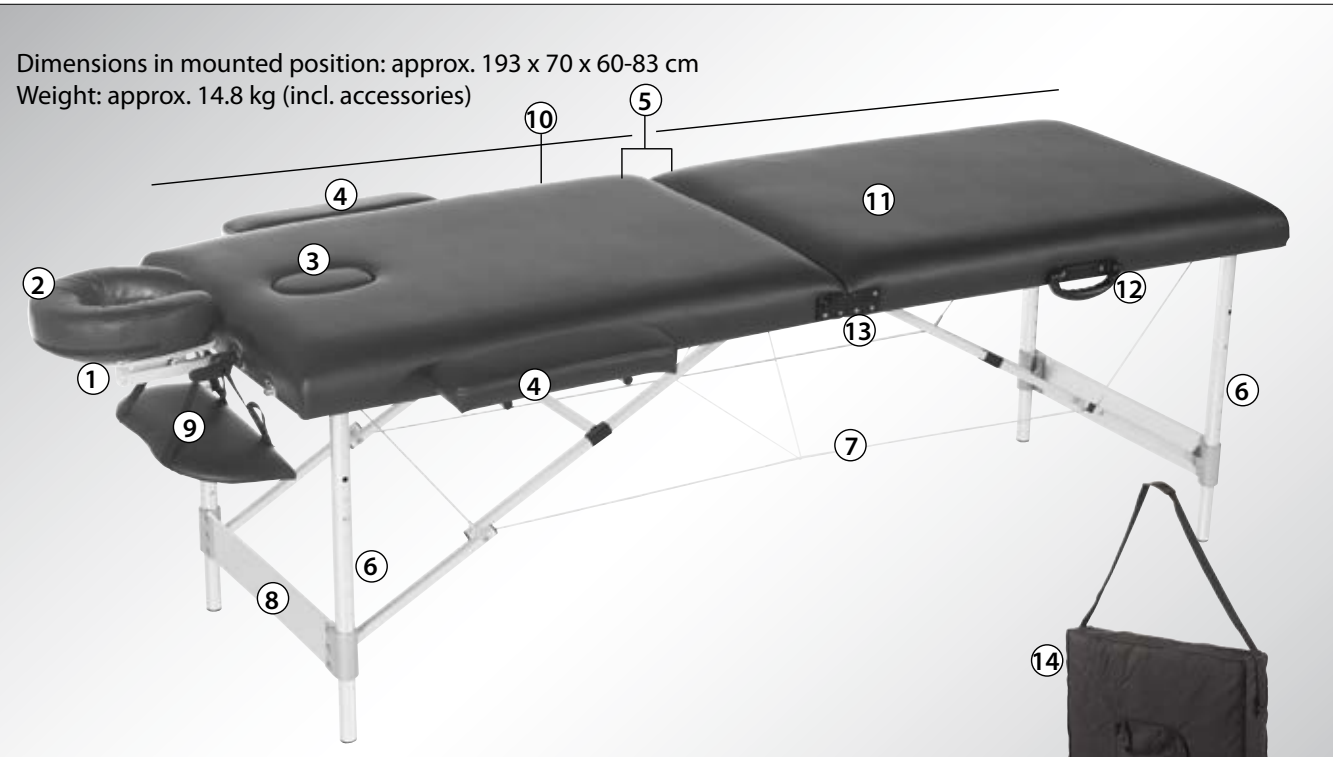- 1 Variable multi-function headrest
- 2 Head cushion
- 3 Face cradle with closing insert
- 4 2 lateral armrests
- 5 Plastic feet
- 6 Height adjustment on legs
- 7 Fastening with stabiliser bar and coated steel ropes
- 8 Aluminium frame
- 9 Handrest
- 10 2-part reclining area
- 11 Cover made of oil- and water-resistant PU material

- 12 Transport handle
- 13 Folding lock
- 14 Carrier bag

A simple folding system ensures fast mounting and demounting of the table.

Important

Carefully read the instruction manual before using the massage table for the first time - the safety instructions in particular. All features are then ready to be used safely and reliably.

Hotline

For technical inquiries regarding this equipment, please use our service hotline or send us an email.

→ Telephone: +49/ 86 85/98 89 80 1

→ Mail: service@royalbeach.de

→ Spare part order: Indicate exact name and order number!

→ Name of equipment: Massage table

Item no.: RB#65581 (Typ 65576)

Mounting

Before beginning to mount the massage table, ensure that the surface is smooth and guarantees a stable position.

For mounting, also ensure sufficient clearance in every direction (min. 2 m).

Fig. 1

Step 1:
Set the closed massage table on the plastic feet at the bottom edge. Loosen the upper and lower clasp and open the table as shown in Fig. 1.
Next, remove all accessories.

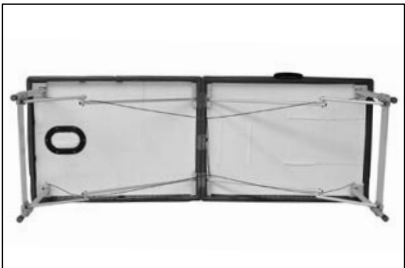
Fig. 2

Step 2:
Stand behind the reclining area. Grab the handle and pivot the two halves of the table carefully outwards to open them completely. The legs should open automatically during this process (see Fig. 2).

Caution: If the legs are blocked when opening the table, it is necessary to find the cause. Do not open when pushing against a resistance.

Step 3:

Lift the massage table at the middle to position it upright. As soon as the reclining area is in horizontal position, ensure that the reclining area is level. If it is not yet completely level, check and adjust the ropes or gently push the reclining area down, if necessary.

Step 4 :

Lock the lateral folding lock on both sides (see Fig. 3).

Fig. 3

Demounting

Unlock the lateral folding lock on both sides (see Fig. 4). Grip the massage table at the middle and tilt it gently until it stands on the plastic feet at the side.
Next, ensure that the stabiliser bars can be folded, that the steel ropes are exposed and that the back rest is folded. Moreover, the accessories must not cause a blockage.
Finally, fold the table and close both clasps.

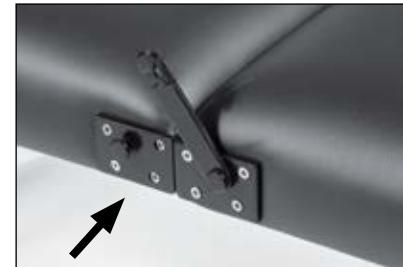
Fig. 4

Installing the accessories

Headrest

The headrest is composed of a crescent-shaped headrest base and a crescent-shaped face cushion. Insert the headrest assembly into the intended holes (1) at the front of the table. In order to adjust the headrest to the individual needs of the user, loosen the lever (2) at the side and move the headrest to the desired tilting angle (3). You lock this position by locking the lever in place (4).

Caution: There is a risk of crushing.

Handrest

The adjustable handrest (5) allows the hands of the user to be supported at the front of the table. Fit the handrest to the headrest bar (6) and adjust the straps until the desired position is reached.


Armrest

Insert both armrests into the intended holes at the sides of the table (7). These additional reclining areas offer yet more comfort.

Adjusting the height

If the massage table still isn't at the desired height, you can adjust it to the individual needs by means of the height adjustment on the legs. It is recommended to lay the massage table first on its left and then on its right side. You can adjust the leg length by pushing both knobs (8) at the same time until the desired position is reached. After setting each leg to the desired height, lift the table back up.

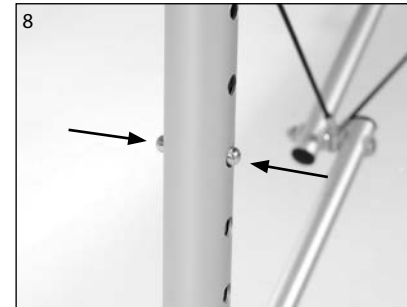Caution: All legs must be at the same height.

Caution: It is recommended to lay a mat under the massage table when laying it on its side.

Notes on the work load

The work load is the sum of the body weight of a lying person and the maximum work pressure of a person. (Example: A lying person has a body weight of 125 kg, the work pressure may not exceed 100 kg since the maximum work load is 225 kg.)

Since pressure from a lateral angle applies a higher load to the table, the maximum pressure from the side may not exceed 50% of the downward pressure.

The person should always be positioned in the middle of the massage table in order to evenly distribute the weight. This ensures stability.

Important: The maximum work weight may never be exceeded. This could result in serious injuries or damage.

Getting off of and onto the massage table

Getting onto the massage table

CAUTION! Risk of injury!

If you lay down falsely onto the massage table, it will be loaded falsely. This may cause the massage table to tilt and you may injure yourself.

- Do not sit on the front or back of the massage table.
- Always position yourself in the middle of the massage table. This is how you lay down safely onto the massage table:
 1. Sit in the middle of the massage table.
 2. Lay down onto the massage table in the desired position. Make sure that you always position yourself in the middle of the massage table in order to evenly distribute the work load.

Getting off of the massage table


CAUTION! Risk of injury!

If you get up falsely on the massage table, it will be loaded falsely. This may cause the massage table to tilt and you may injure yourself.

- Do not sit on the front or back of the massage table when getting up. This is how to sit up safely on the massage table:
 1. Sit up in such a way that you sit in the middle of the massage table.
 2. Get up slowly. Quick movements are not recommended after lying down on the massage table for a longer period since your circular flow must adapt correspondingly.

Storage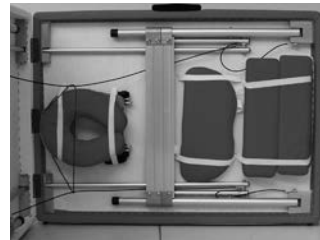

Attach the accessory parts (head rest including facial padding, hand rest as well as both arm rests) by means of the rubber strips to the bottom side of the table. Make sure the table can still be folded and locked. The table must not be folded when under tension. The carrier bag should be used to transport and store the table in order to protect it against damage.

If not in use, the table should be stored in a cool and dry place. Please consider that the surface temperature may increase due to direct sunlight. Thus, do not expose the massage table to direct sunlight. Do not expose the table to extreme temperatures (very high or low temperatures) or humidity.


Maintenance instructions

Cleaning:

For daily cleaning, we recommend to remove dirt with water and a mild soap. Subsequently, dry the table with a soft and lint-free cloth. Slightly heavier contamination may be removed by means of a mild, non-abrasive cleaning agent. Spray a damp cloth with the cleaning agent and carefully wipe the stains away. Then rinse with warm water and dry with a lint-free cloth. A professional cleaning is recommended for tenacious stains.

Important: Never use rough or abrasive cleaning agents, chemicals or impregnating agents in order to clean the table. This could damage the cover and the padding.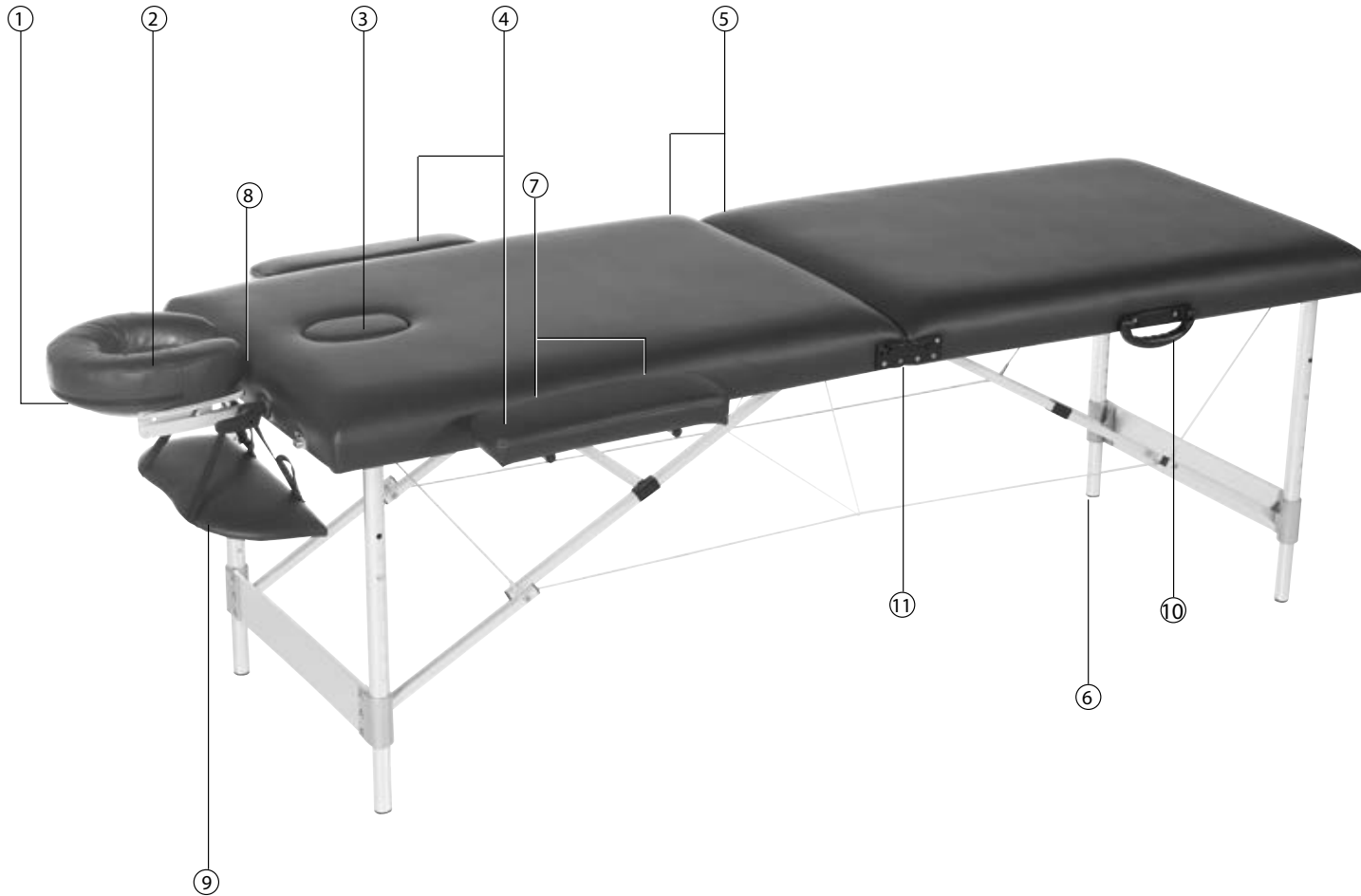

- 1 Headrest base
- 2 Head cushion
- 3 Closing insert
- 4 Armrests
- 5 Plastic feet
- 6 Plastic end caps
- 7 Plastic jacket armrest
- 8 Plastic jacket headrest
- 9 Handrest
- 10 Transport handle
- 11 Folding lock

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Spielwaren und Sportartikel Vertriebs GmbH