Anleitung

15003000

pedalo[®] - Fasziensticks

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User manual

15003000 pedalo® - FasciaSticks

Before using the device for the first time (or before starting the assembly), please read the instructions completely and keep it.

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info@pedalo.de
www.pedalo.de

Warning!

Warning! These Pedalo® FasciaSticks are only intended for therapeutic use by qualified therapists and are meant to support their treatment. The use of the FasciaSticks for treating fascia requires a comprehensive knowledge only a therapist has acquired in the framework of his education. Too much pressure can cause vascular injury. Therefore you should always work only with well-dosed force and only apply on body areas on which vascular injuries may not happen.

Warning! The packing is not part of the product and have to be disposed.

Keep unfastened small parts out of the reach of babies and young children – RISK OF SUFFOCATION! Pedalo® FasciaSticks are exclusively conceived for use by adults.

Warning! The equipment is only to be used for the purposes described in this user manual.

Warning! Before each use check the connectors of the FasciaSticks. In case of any visible defects don't use them anymore. Protect the sticks from permanent moisture. After spraying with disinfectants, please rub dry with a cloth immediately.

Cleaning purposes: Wipe over with a moist (not wet!) cloth. After cleaning rub dry with another cloth. Never use any abrasive detergents. On request use a disinfectant wipe (to this please observe the manufacturer's information).

The main aim of a fascial therapy is to relieve tensions and pain in order to bring about sustained improvement in the range of movement.

The cause of much pain and movement restrictions are agglutinations in the connective tissue. With the Pedalo® FasciaSticks there is a new helpful tool available, with which the therapist is able to treat the fascia in an effective and targeted way. The Pedalo® FasciaSticks are in so far a facilitation for the therapist when treating, as he needs less own power to keep up pressure on the treatment point for a longer time. Also he preserves his own hands and joints when working with Pedalo® FasciaSticks. Thanks to the balls the sticks are very comfortable and ergonomic in handling. The attractive treatment tool made of the natural material wood is especially geared to physiotherapists' and masseurs' requirements.

Specification: Solid beech, surface clear lacquered. 5-part set, consisting of: 3 Trigger Pointer 25-30-40 mm, Trigger Stick with pointer, Fascia Roller 310 mm. Supplied in a wooden box (34.7x21.7x7.0 cm) with a soft inlay. Weight of the sticks (5 pcs.) w/o box 0.4 kg. Weight with box 1.35 kg

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Holz-Hoerz GmbH