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<p>Product information</p> <p><b>Item 22220000</b> <b>Pedalo® Knee Fit</b></p>	<p>Please read before using the product</p> <p><b>CE</b></p>
<p>Item picture see</p>  <p style="text-align: right;">above</p>	<p><b>Product content:</b></p> <ul style="list-style-type: none"> <li>✓ 1 Knee Fit</li> <li>✓ Incl. 1 pad self-adhesive as cut underside side parts</li> </ul> <p>For more details see <a href="http://www.pedalo.de">www.pedalo.de</a></p> <p><b>Product features:</b></p> <ul style="list-style-type: none"> <li>✓ Stand surface in the inclined position adjustable in 4 angular degrees</li> <li>✓ Baseboard stop upholstered</li> <li>✓ Hip width standing surface with anti-slip surface</li> </ul>
<p><b>Special notes:</b></p> <ul style="list-style-type: none"> <li>• Knee fit is used to stretch your leg muscle chains with one or both legs in different angular positions of the foot towards the lower leg.</li> <li>• Warning. Before stretching, appropriately warm up the muscles to be stretched.</li> <li>• Warning. In case of existing injuries in the foot and leg muscles, use only under the supervision of a therapist.</li> <li>• Kniefit is not designed for jumping exercises.</li> <li>• To adjust the angle of the footprint, it is pulled backwards out of the grid and pushed completely forwards into the stop at the desired grid. Adjustment takes place in 4 degrees of angle.</li> <li>• The base has an anti-slip coating.</li> <li>• Before each use, please check that the footprint is properly engaged.</li> <li>• Train with shoes</li> </ul>	
<p><b>Important information and warning hints:</b></p> <ul style="list-style-type: none"> <li>• Warning not suitable for children under 12 years.</li> <li>• Use only under adult supervision.</li> <li>• Warning! Packaging is not part of the articles and must be disposed of before use.</li> <li>• Warning! Only use the device for the purposes described in these operating instructions.</li> <li>• Wearing of protective clothing is recommended.</li> <li>• Warning! Only set up the device on a level, even surface. If the floor is sensitive, place a mat underneath.</li> <li>• Plastic bags and plastic foils do not belong in the hands of babies and small children - danger of suffocation!</li> <li>• Replace worn parts immediately with an original spare part.</li> <li>• The product is made of different woods. Among other things, natural birch, beech wood and multiplex. Deviations in color, structure, grain or small stone inclusions testify to the authenticity and are not quality-reducing.</li> <li>• Maintenance and care: Can be wiped with a damp cloth. Do not use solvent-based, acidic or abrasive cleaners. Protect device from moisture.</li> <li>• The maximum load capacity is 150 kg</li> </ul>	
<p><b>Assembly:</b> No assembly required. The training device is delivered ready for use.</p>	