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Product information Item 20270000 Pedalo® Stabilizer Therapy	Please read before using the product  20270000 Pedalo® Stabilizer Therapie D-72525 Münsingen Dottinger Str. 71 www.pedalo.de Made in Germany pedalo® EAN-Code 4260286360571 This is a sports equipment. Recommended for ages 14+
	Scope of delivery ✓ 1 wooden board ✓ 1 wooden standing platform with rubber feet ✓ 2 wooden strips ✓ 1 metal frame (4-part) ✓ 1 set of screws ✓ 1 anti-slip mat ✓ 1 exercise poster Specification Birch multiplex, clear lacquered. 1 standing board 60x35cm with all-round grid holes for accessories. 1 standing platform 95x56cm with 4 rubber feet for easier access. 2 movement limiters with 4 rubber fixing rings. Powder-coated metal frame in anthracite, upper holding brackets covered with wipe-clean soft grip. 1 set of screws (ropes, springs, carabiners ...) and 1 anti-slip mat 48x29cm. Assembled 95x56x107.5cm. Weight capacity up to 120kg. 17.3kg. For more details see www.pedalo.de
Special notes • The Stabilizer Therapy (hereinafter also referred to as the device, product or item) serves to improve the sensory control of posture and movement. Even the smallest imbalances and instabilities in the musculoskeletal system are identified and treated thanks to the high sensitivity of the 3-dimensional standing surface. Therapy and training on the Pedalo Stabilizer are highly effective because its mode of action increases the input information of the proprioceptors and receptors. This leads to stimulation and improved coordination of the intersegmental muscles. • The high brackets on the frame of the stabilizer serve as a handrail. If required, the device can also be positioned in front of a wall bar or similar to provide additional support. • The Stabilizer may only be used by one person. • Warning! This device is a very demanding one! When using the device (especially during the first attempts at exercise and/or when doing exercises in a standing position), a particular degree of caution, attention, circumspection and a high level of motor skills are required. If in doubt, only use the device with assistance and/or appropriate protective equipment. • Attention: In case of existing injuries (especially in the foot and leg muscles), use only under the supervision of a therapist. • Before each use, check the screw, plug-in and suspension connections as well as the device as a whole for damage. Never use the stabilizer if it is visibly damaged! • The springs and metal parts must be checked for bending every 4 weeks. • Replace the springs at least every 2 years, depending on the intensity of use. We recommend always replacing all the springs. • The Anti-Slip Mat supplied provides a safe standing on the device. It should only be placed on the wooden plate for the duration of use and removed from the stand board for storage or cleaning (it can be cleaned with a damp cloth). • The maximum weight capacity is 120 kg with a static and even load on all 4 springs. With dynamic or one-sided loading, the value is reduced considerably.	
Important information and warnings • Not suitable for children under 14 years. Use only under adult supervision. • Caution! Packaging is not part of the item and must be disposed of before use. • Warning! Plastic bags and foils do not belong in the hands of babies and small children – risk of suffocation! Keep out of reach of babies and small children! • Caution! Use the device exclusively for the purposes described in these instructions (and any other pedalo product information enclosed with the product). • Warning! Long cord – risk of strangulation. • Warning! The device may only be used for balance exercises. NOT suitable for jumping exercises! • Warning! Only use the device on level ground with even and firm surfaces. On sensitive floors, place a mat underneath. • After longer or frequent use, check that the screws (and installed parts) are tight and retighten if necessary. • Replace worn parts immediately and only with original spare parts. • The product is made of different woods. Among others from natural birch, beech, and multiplex. Deviations in color, structure, grain, or small knots inclusions prove the genuineness of the material and therefore is no reduction in quality. • Maintenance and care: Can be wiped with a damp cloth. Do not use solvent-based, acidic, or abrasive detergents. Protect item from damp and wet. In addition, the above-mentioned visual inspections and maintenance intervals must always be observed.	

Assembly of the frame:

- Step 1: Connect the two lower frame parts (brackets with the black plastic feet) with the wooden movement limiters. To this, place the movement limiters (with the outermost large holes) onto the brackets. Then attach the two upper frame brackets to the lower parts. Slight scratches on the plug-in connections may be the result of a careful goods inspection (testing and checking the plug-in connections).
- Step 2: Each pipe connection is now firmly connected with a through-threaded pin with screwed-on sleeve nuts. The second provided Allen key is used to counter-hold when tighten the screw.
- Step 3: The scope of delivery includes 4 attachment loops. Please check the knots of the loops and tighten them if necessary (the hard rope ends should stand out completely behind the knot). Then insert each on piece of attachment loop into each corner hole of the plate form below. Each rope loop can bear a maximum load of 200 kg.

Step 1

Step 2

Step 3

Assembly of the ropes and springs:

Prepare the springs as shown on the right side:

- Hook the spring to the 5th loop from below into the 'Isilink-rope'.
- Pull the short end of the rope through the spring from top to below.
- Hook the carbine into the bottom loop of the rope.

Now thread the rope from below through the small holes of the movement limiter and attach it to the metal hook of the upper handle bar.

Hang all 4 springs in the tube frame in this way.

Assembly of the standing board:

Variant 1 – without spring function

The board can now be attached directly to the long 'Isilink-ropes' by using the carbiners and the small (orange colored) rope loops as connection parts.

Thus, the board only provides horizontal swinging movements in all directions.

Variant 2 – with spring function


The board can also be attached directly to the additional springs and the long 'Isilink-ropes'. Both, 'Isilink-ropes' and the springs are now hooked in the carbiners. This is to avoid distension of the springs and prevents the board from touching the floor.

Now when exercising, the board provides horizontal and vertical movements.

How to adjust the limiters:

The swing area of the stand platform can be varied in steps by parallel pushing or pulling the movement limiters. Thereby it is not necessary to remove the rubber rings from the limiters.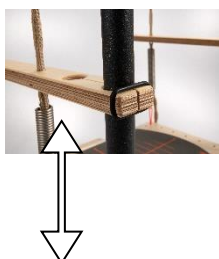

Accessory Standing Platform

Offers a first step to have an easier entrance onto the Stabilizer. Furthermore, used this way, the frame of the Stabilizer can also be used as a holding rack for exercises with other training equipment on request.

Optional accessories (not included in delivery):

Pedalo® Softpads Item no. 144056, enforce the training effect. Special foam, 2 pcs. 30x14x2.5cm

Pedalo® Footboards Seesaw Item no. 653101, increase the level of difficulty and aim on foot and leg axis stability. Can be attached to Pedalo products with grid holes but are also suitable to be used individually on the floor (retractable pins).