Art.-Nr. 1137402

## SECUMAR 9 S

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Tel: +49 4103 125-0 • Fax: +49 4103 125-100

## Instructions for use

### SECUMAR 9S Buoyancy collar for bathing therapies

#### Description

The **SECUMAR 9S** is worn around the neck and under the chin. Correctly inflated, the collar has sufficient buoyancy to maintain the wearer in a stable position in water. If the wearer is lying backwards, it acts as a head support, keeping mouth and nose above water.

The **SECUMAR 9S** buoyancy collar is available in three collar sizes. For easier identification each size has different coloured label.

Size	Collar size	Order-No.	Label colour
Small	28 - 32 cm	12278	yellow
Medium	33 - 38 cm	12279	green
Large	39 - 43 cm	12280	blue

#### Important note

This buoyancy collar is a buoyancy aid in the sense of a "Personal Protective Equipment" (PPE). PPE's of this category may only be used under supervision! They are not lifejackets and do not offer protection against drowning. Furthermore, it will only fulfill this function if it has been fully inflated and is correctly worn. Take care to choose the correct collar size to assure correct head support under the chin.

The buoyancy collar may only be used in the named fields of application.

Read the instructions carefully and familiarise yourself with the device before use. Instruct children properly.

Make sure the device is ready for use before putting on (refer par. "Inspection"). Only use under supervision!

#### Fields of application

Purpose of the SECUMAR 9S is to give weak swimmers, disabled or older people, patients doing a rehabilitation help and safety in water. Possible fields of application for the SECUMAR 9S buoyancy collar are swimming therapies, rehabilitation and domestic sectors such as treatment in kinotherapeutic baths, medicinal or cleansing baths or underwater massage in bathtubs.

Kinds of baths, chemical composition:

The SECUMAR 9S can be used in chlorine baths. Material impairs caused by bathing additives are unknown. Given the multitude of swimming additives/bath salts at present in use, however, impairs cannot be excluded completely. Oily additives may cause permanent residents and discoloration.

#### Contraindications

Do not use the buoyancy collar if there will be contact to open wounds or injured skin. The buoyancy collar may not be used with patients that are not able to take a safe back inclined position in water.

## **Donning**

Ensure, that the buoyancy collar is the correct size before using. Place the open collar around the neck so that the oral valve is on the upper left-hand side (viewed from the wearer). Close the snap fastener.

## **Inflation**

Inflate the buoyancy collar via the oral valve. Therefore remove the dust cap from oral inflation valve. Put the dust cap back in position after inflation.

## **Handling after use**

After use the buoyancy collar should be thoroughly rinsed and hung up to dry. Deflate via the oral valve turning the dustcap around and depress the valve insert. Smooth out the air from the collar.

## **Inspections**

The buoyancy collar should be checked at regular intervals. The following inspections show the readiness of the buoyancy collar.

### **Leakage test**

Fully inflate the buoyancy collar via the oral valve and leave inflated for approx. 16 hours. If the collar shows signs of deflation after this period, it must be replaced.

### **Testing the oral valve**

Depress the valve insert with the turned dust cap. Remove dustcap, the valve has to close properly.

## **Cleaning instructions**

Cleaning can be best done while the collar is inflated. Rinse with fresh water and hang up to dry. If necessary the collar can be cleaned with 0.5% disinfecting solution (Sterilium).

If the SECUMAR 9S is used in hospitals, rehabilitation centres or similar facilities the valid hygienic regulations have to be observed.

The collar has to dry completely after cleaning.

## **Storage**

Always store the buoyancy collar in a dry condition. Protect from permanent sunlight.

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GmbH u. Co.