SISSEL[®] Massage Stuhl

The natural way of Sweden

Art.Nr. 2125839

**Gebrauchsanweisung
Operating manual
Mode d'emploi
Instrucciones de usuario
Gebruiksaanwijzing
Istruzioni per l'uso
Bruksanvisning**

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Congratulations on the purchase of your new portable massage chair. Your new chair has been skillfully crafted to exacting standards, utilizing high quality components and materials. With proper use and care, as outlined in this user's guide, your chair should provide a longtime of reliable performance.

To insure the safe and correct use of your chair, please take a moment to read this entire guide prior to using your chair.

Precautionary instructions:

1. Maintain the chair periodically. Make sure all screws are securely fastened and the metal frame has not deteriorated or frayed.
2. Do not overload the chair beyond the recommended weight limit. This could result in serious injury to the user.
3. Do not use alcohol based cleaners when cleaning the vinyl. This could cause damage to the vinyl material. Use warm soapy water and wipe dry.
4. Make sure when using the chair that all four legs are on a solid surface and at the same level.

Chair Set Up:

Place the chair on its side, release the lever, partially open the chair and remove the accessories. Opening the leg, headrest and armrest, the legs should open automatically and then press down in the middle of the chair to make sure the chair lays flat (Figure 1).

Use of accessories:

We provide optional accessories for you. Please read the instruction carefully before use.

Headrest:

The Headrest consists of a crescent Headrest Base and a crescent Face Pad. It will give your client comfortable head and face support. The headrest can be adjusted easily. Simply release the lever, set to desired position and secure the lever tightly. (Figure 2).

Adjustable Armrest Board:

The adjustable armrest board is a good device to support your client's arms at the front of the chair. The armrest board can be adjusted at various heights (Figure 3).

Massage Chair is fully adjustable:

Seat height (Figure 4), chest pad (Figure 5), armrest (Figure 6) and head support (Figure 7) which helps your patient to be fully relaxed for maximum massage effect. The ergonomically molded chest pad prevents from unpleasant pressure at your patients arms so that they can be positioned relaxed on the armrest.

Maximum working weight for the chair: 225 kg

The working weight is the combined weight of the client and the maximum amount of downward pressure being applied by the therapist. For example: for a client weighing 140 kg, the maximum downward pressure that can safely be applied is 85 kg.

DO NOT EXCEED REFERRED MAXIMUM WORKING WEIGHT! (Never exceed the load limits of the chair; doing so may result in serious injury and/or damage).

Storage:

Folding all the accessories at their original position (Figure 8). The carry case should be used when transporting or storing the chair to help protect the chair from damage. When not in use, store the chair in a cool dry place and away from direct sunlight. Do not expose the chair to extreme (hot or cold) temperatures or moisture.

Chair Care:

Upholstery care:

Daily Cleaning: Remove daily dirt and grime from the chair with a mild soap and warm water solution and dry with a soft lint-free cloth.

Disinfecting: Disinfect the chair as required with an isopropanol based hospital grade disinfectant cleaner (approved for use on polyurethane vinyl).

Stain Removal: some minor stains can be removed with a mild non-abrasive cleaner. Spray a damp cloth with cleaners and gently wipe to remove stain, rinse thoroughly with warm water and dry with a soft lint-free cloth. More stubborn stains may require professional cleaning.

Note: Never use harsh or abrasive cleaners or chemicals to clean the chair. Doing so may damage the upholstery. Avoid using any protectants as they may harm the vinyl.

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