SPINEFITTER

BACK IN BALANCE

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1. GENERAL INFORMATION

Read the following instructions for use carefully and observe the safety instructions and warnings. The SPINEFITTER by SISSEL® Extension Kit is used to extend the SPINEFITTER by SISSEL® by 2 pairs of balls using a simple, secure click system. This makes the set ideal for tall users over 180 cm. The product can also be used without the Connector as a stand-alone device to support training, improve spinal and joint mobility, relieve tension, strengthen deep muscles and for balance exercises. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or how to avoid them. The therapeutic use of the SPINEFITTER by SISSEL® Extension Kits, e.g. in physiotherapy or rehabilitation treatment, should preferably be carried out by or under the guidance of professional users. Autonomous training by non-professional users and medical laypersons is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists. SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these instructions for use. The instructions for use must be kept carefully and within easy reach and must be enclosed if the product is passed on. Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

2. PRECAUTIONARY INSTRUCTIONS

- ! Check the condition of the product before each use. If damaged, the function of the product and safety are no longer guaranteed.
- ! Do not use the SPINEFITTER by SISSEL® Extension Kit in the immediate vicinity of heat sources (e.g. radiators, hotplates, open flames, flying sparks) and avoid direct sunlight.
- ! Keep the exercise area free from sharp and pointed objects.
- ! Exercise only on dry, non-slip, straight surfaces and not near stairways, balustrades, edges, etc.
- ! Train exclusively barefoot or wear socks with a non-slip base. Do not train with shoes and make sure you wear suitable sports clothing.
- ! Choose a training programme that suits your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk.
- ! Do not insert fingers or other body parts into the openings of the training device.
- ! Do not bend the individual parts of the SPINEFITTER by SISSEL® Extension Kit. Only use and store the extension kit on a flat surface and do not place any objects on top of it.
- ! Only use accessories provided by the manufacturer.
- ! Carrying straps must be removed before use.
- ! The product and its accessories are not toys and must be kept out of the reach of children. Do not allow children to play with it. Risk of injury and strangulation!

3. PRODUCT INFORMATION

Scope of delivery

SPINEFITTER by SISSEL® Extension Kit: 1 x SPINEFITTER by SISSEL® Mini,
1 x SPINEFITTER by SISSEL® Connector, 1 x instruction manual

SPINEFITTER by SISSEL® Extension Kit Set of 5: 5 x SPINEFITTER by SISSEL® Mini,
5 x SPINEFITTER by SISSEL® Connector, 1 x instruction manual

Technical specifications

Dimensions (L x W x H):	Mini: approx. 12 x 12 x 6 cm Connector: approx. 18 x 5 x 4 cm
Material:	Mini: Polyurethane Connector: Polyurethane
Max. User weight:	Mini: 150 kg Connector: 150 kg

Note: The product is made from a high-quality polyurethane foam. Please note that this naturally has a slightly structured, irregular surface. This does not represent a product defect and has no influence on the use or cleaning of the product.

Intended use

Use for general training

The SPINEFITTER by SISSEL® Extension Kit in combination with the SPINEFITTER by SISSEL® for tall users over 180 cm is an innovative training device that combines relaxation and effective training. Can be used in private or professionally guided individual or group training, the combination or the SPINEFITTER by SISSEL® Mini alone is suitable for a variety of exercises thanks to its unique shape and is an ideal support for releasing tension and adhesions and improving spinal and joint mobility. By simply placing the body intuitively on the SPINEFITTER by SISSEL® Mini, a loosening, e.g. of the shoulder girdle, can already take place and a stimulation of the energy points along the spine can take place. This makes it the perfect everyday partner to bring the body into balance and promote mobility.

Therapeutic use

The SPINEFITTER by SISSEL® Extension Kit, in combination with the SPINEFITTER by SISSEL® for large users, or the SPINEFITTER by SISSEL® Mini alone are used as a portable therapeutic aid for active loosening of myofascial tissue, loosening and relaxation of muscles close to the spine, individual massage, stimulation of blood circulation and metabolism as well as mobilisation of the spine in the area of the vertebral bodies. In addition, the product can be used as part of physiotherapeutic therapy programmes and as a therapeutic aid in the standing position for functional balance exercises, psychomotor training, for training body awareness and for fall prevention exercises.

Indication

- Chronic shoulder-neck tension
- Neck tension and neck pain
- Tense, hardened muscles near the spine
- Movement restrictions of the cervical spine, thoracic spine, shoulder joints
- Diffuse pain in the shoulder, neck, spine, lumbar spine
- Cervical, thoracic and lumbar spine syndrome
- Therapeutic aid for physiotherapy, osteopathy, sports medicine, rehabilitation
- Supporting aid for measures to improve posture, body awareness, body perception, movement sequences, balance, general coordination, psychomotoric skills, coordination
- Addition to and optimisation of the usual fitness programme

Contra indication

- Lack of compliance (e.g. due to fear or pain).
- Acute injuries (open wounds, soft tissue injuries, unstable fractures in the spine, pelvis, extremities, etc.)
- Acute radiating pain, paraesthesia, movement disorders (consult specialist or physiotherapist)
- Osteoporosis
- Intervertebral disc problems
- Acute movement disorders
- Pronounced sensory perception disorders
- Usage by children and adolescents, during pregnancy or in case of general health problems only under professional supervision and after professional consultation

 Risk of injury! Use the SPINEFITTER by SISSEL® with the Extension Kit in case of pronounced sensory perception disorders, unstable fractures, open wounds, acute injuries and movement disorders only after prior consultation with a doctor.

 Risk of injury! Only use the SPINEFITTER by SISSEL® with the Extension Kit if you can ensure correct performance of the exercise recommendations described. Instruction and supervision by qualified personnel is recommended.

Regulatory requirements

The SPINEFITTER by SISSEL® Extension Kit by SISSEL® GmbH fulfils the regulatory requirements of Regulation (EU) 2017/745 on medical devices.

 All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

Accessories

The SPINEFITTER by SISSEL® Extension Kit may only be used with accessories offered or approved by the manufacturer (see www.spinefitter.com).

4. APPLICATION

1. Check the packaging and the product for any damage on delivery.
 Never use a damaged product.
2. Check the product for damage and contamination before each use.
3. Use the product as intended. Remove the carrying strap before use!
4. Warm up before exercising.
5. Exercise according to the exercise instructions (see www.spinefitter.com) or as instructed by professionals.

Note: After use, cleaning and possible disinfection measures may be necessary (see Care & Hygiene).

5. CARE & HYGIENE

This product is suitable for reuse by additional persons. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed to ensure a safe operation. If required, clean the SPINEFITTER by SISSEL® Extension Kit with a soft cloth and mild soap and water solution. If required due to change of user, the surface of the SPINEFITTER by SISSEL® Extension Kit can be disinfected by wiping with an alcohol free solution. Disinfection products that contain caustic substances (e. g. acids, alkalis) should be avoided as they can damage the surface of the product.

-  Disinfection measures are to be carried out exclusively with suitable disinfectants that have been tested for effectiveness.
-  Disinfectants that can damage the materials of the product must be avoided.
-  The application instructions of the disinfectant manufacturer must be observed.

The use of unsuitable disinfection agents or agents that have not been tested for effectiveness or their incorrect use will invalidate the warranty and guarantee claims.

6. DISPOSAL & ENVIRONMENT

The product and the accessories supplied can be disposed of with household waste.
No special types of packaging are used. The respective valid disposal guidelines must be observed.

7. WARRANTY

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL® GmbH have been handled, treated or modified by others without consent or if the instructions for use have not been followed. Operators of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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SPINEFITTER by SISSEL® Extension Kit: 1 x SPINEFITTER by SISSEL® Mini,
1 x SPINEFITTER by SISSEL® Connector, 1 x manuel d'instruction

SPINEFITTER by SISSEL® Extension Kit Set of 5: 5 x SPINEFITTER by SISSEL® Mini,
5 x SPINEFITTER by SISSEL® Connector, 1 x manuel d'instruction

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SISSEL® GmbH · Bruchstrasse 48