# SPINEFITTER

BACK IN BALANCE

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# SPINEFITTER BY SISSEL® TRIGGER TOOL

## 1. GENERAL INFORMATION

Read the following instructions for use carefully and observe the safety instructions and warnings. The SPINEFITTER by SISSEL® Trigger Tool is an attachable module for expanding the SPINEFITTER by SISSEL® and SPINEFITTER by SISSEL® Mini. It allows targeted, punctual pressure to be applied to trigger points (points of pain), hardenings, tensions and myogelosis. Especially paravertebral trigger points, i.e. lying next to the spine, can be addressed particularly effectively with the Trigger Tool. The extension thus offers a simple way to independently and specifically release trigger points and hardenings along the spine. In combination with the SPINEFITTER by SISSEL® Mini, additional points on the thigh, shoulder, back and neck can be reached (e.g. leaning against the wall). This makes the module the perfect addition to the daily training routine. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or how to avoid them. The therapeutic use of the SPINEFITTER by SISSEL® Trigger Tool, e.g. in physiotherapy or rehabilitation treatment, should preferably be carried out by or under the guidance of professional users. Autonomous training by non-professional users and medical laypersons is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists. SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these instructions for use. The instructions for use must be kept carefully and within easy reach and must be enclosed if the product is passed on. Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

## 2. PRECAUTIONARY INSTRUCTIONS

- ! Check the condition of the product before each use. If damaged, the function of the product and safety are no longer guaranteed.
- ! Do not use the SPINEFITTER by SISSEL® Trigger Tool in the immediate vicinity of heat sources (e.g. radiators, hotplates, open flames, flying sparks) and avoid direct sunlight.
- ! Keep the exercise area free from sharp and pointed objects.
- ! Exercise only on dry, non-slip, straight surfaces and not near stairways, balustrades, edges, etc.
- ! Train exclusively barefoot or wear socks with a non-slip base. Do not train with shoes and make sure you wear suitable sports clothing.
- ! Choose a training programme that suits your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk.
- ! Do not insert fingers or other body parts into the openings of the training device.

- ! Use the Trigger Tool only in combination with the SPINEFITTER by SISSEL® or the SPINEFITTER by SISSEL® Mini.
- ! Do not bend the Trigger Tool. When releasing, do not grip the humps of the product, but in the middle at the level of the SISSEL® logo.
- ! Do not stand or support yourself on the product. In combination with the SPINEFITTER by SISSEL® only use lying down or with the SPINEFITTER by SISSEL® Mini leaning against the wall and with a flat back!
- ! Carrying straps must be removed before use.
- ! Only use accessories provided by the manufacturer.
- ! The product is not a toy and should be kept out of the reach of children. Do not allow children to play with it. Risk of injury and strangulation!

### 3. PRODUCT INFORMATION

#### Scope of delivery

1 x SPINEFITTER by SISSEL® Trigger Tool, 1 x instruction manual

#### Technical data

Dimensions (L x W x H):	approx. 12 x 4,5 x 7 cm
Material:	Polyurethane
Weight:	approx. 55 g
Max. User weight:	150 kg

**Note:** The training device is made from a high-quality polyurethane foam. Please note that this naturally has a slightly structured, irregular surface. This does not represent a product defect and has no influence on the use or cleaning of the product.

#### Intended use

##### Use in therapy and training

The SPINEFITTER by SISSEL® Trigger Tool is used in therapy for paravertebral muscle hardening and tension, primarily for use on the SPINEFITTER by SISSEL®. The product is positioned on the SPINEFITTER by SISSEL® so that it can have an effect on the muscle hardening and pain points (so-called trigger points), which are located next to the spine (e.g. between the shoulder blades). With the Trigger Tool, tensions all over the body can be worked on by yourself. Loosening of hard-to-reach areas of the body such as the back, shoulders, calves or buttocks before and after training can also be achieved. By simply placing the body intuitively on the SPINEFITTER by SISSEL® with the Trigger Tool, a selective loosening of the muscles can already take place. The intensity of the pressure can be varied and regulated by your own body weight. The Trigger Tool is therefore the

perfect extension of the SPINEFITTER by SISSEL® and the SPINEFITTER by SISSEL® Mini to specifically treat hardening and pain points of the muscles near the spine and to prevent the formation of new myogeloses.

### Recommendation for use

Before the trigger point treatment, the corresponding body part can be prepared with the help of heat application, e.g. with the SPINEFITTER by SISSEL® Linum / Linum Pro. Then place the Trigger Tool on the SPINEFITTER by SISSEL® at the height of the body part to be treated. Now place it on the SPINEFITTER by SISSEL® so that the Trigger Tool lies exactly on the painful points on the body. Adjust the pressure according to the intensity of the pain. Alternatively, the Trigger Tool can also be used in combination with the SPINEFITTER by SISSEL® Mini while standing. Vary the pressure according to the intensity of the pain. Apply the treatment slowly and in a controlled manner. Continue breathing slowly and calmly. Initially, a short period of application is sufficient; the pressure should be easy to bear. In the case of static application (i.e. simply lying down) until the pain has clearly decreased or for a maximum of 120 seconds! With dynamic application (e.g. rolling the upper body up and down) work into the trigger points 3 to 6 times.

**Note:** There must be no excessive pain or haematoma during and after application!

**Note:** The prescribed application time should not be exceeded. Pay attention to an adapted acclimatisation phase with shortened application duration, increase this slowly and continuously.

**Note:** Avoid too strong, intense pressure!

**Note:** Stop the exercise immediately if you feel severe pain or discomfort!

### Indication

- Local, paravertebral pain
- Myogelosis, localised muscle stiffness, painful muscle points along the spine
- Radiating pain (consultation with attending physician)
- Acute pain
- Acute and/or chronic tension
- Therapeutic aid in physiotherapy, osteopathy, sports medicine, rehabilitation
- Extension and optimisation of the usual fitness programme

### Contraindication

- Lack of compliance (e.g. due to fear or pain)
- Acute injuries (open wounds, soft tissue injuries, unstable fractures in the spine, pelvis, extremities, etc.)
- Acute infections, inflammations, fever, status after irradiation
- Blood coagulation disorders, haematomas
- Blood-thinning medication (consult specialist or physiotherapist)
- Acute radiating pain, sensory disturbances, movement disorders (consult specialist or physiotherapist)
- Osteoporosis
- Use in the area of malignant tumours
- Intervertebral disc problems
- Acute movement disorders
- Use by children and adolescents, during pregnancy or in case of general health complaints only under professional supervision and after professional consultation

**⚠ Risk of injury!** Only use the SPINEFITTER by SISSEL® Trigger Tool in cases of pronounced sensory perception disorders, unstable fractures, open wounds, acute injuries and movement disorders after prior consultation with a doctor.

**⚠ Risk of injury!** Only use the SPINEFITTER by SISSEL® Trigger Tool if you can ensure correct performance of the exercise recommendations described. Instruction and supervision by qualified personnel is recommended.

### Regulatory requirements

The SPINEFITTER by SISSEL® Trigger Tool by SISSEL® GmbH complies with the regulatory requirements of Regulation (EU) 2017/745 on medical devices.

 All serious incidents occurring in connection with the product must be reported immediately to the manufacturer and the competent authority.

### Accessories

The SPINEFITTER by SISSEL® Trigger Tool may only be used with accessories offered or approved by the manufacturer (see [www.spinefitter.com](http://www.spinefitter.com)).

## 4. APPLICATION

1. Check the packaging and the product for any damage on delivery.

 Never use a damaged product.

2. Check the product for damage and contamination before each use.

3. Use the product as intended.

4. Warm up before exercising.

5. Exercise according to the exercise instructions (see [www.spinefitter.com](http://www.spinefitter.com)) or as instructed by professionals.

Note: Cleaning and disinfection may be necessary after use (see Care & Hygiene).

## 5. CARE & HYGIENE

This product is suitable for reuse by additional persons. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed to ensure a safe operation. If necessary, clean the SPINEFITTER by SISSEL® Trigger Tool with a soft cloth and mild soap solution. The surface of the Trigger Tool can be wipe-disinfected with a non-alcoholic preparation if necessary and when changing users. Test on an inconspicuous area beforehand. Disinfection products containing corrosive substances (e.g. acids, alkalis) should be avoided as these can damage the surfaces.

 Disinfection measures should only be carried out with suitable disinfectants that have been tested for effectiveness.

 Disinfectants that can damage the materials of the product should be avoided.

 The application instructions of the disinfectant manufacturer must be observed.

The use of unsuitable disinfectants or disinfectants that have not been tested for effectiveness or their incorrect use will invalidate the warranty and guarantee.

## 6. DISPOSAL & ENVIRONMENT

The product and the accessories supplied can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be observed.

## 7. WARRANTY

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed. Users of this product must observe the regulatory requirements for medical products applicable at the place of use.

# SPINEFITTER BY SISSEL® TRIGGER TOOL

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1 x SPINEFITTER by SISSEL® Trigger Tool, 1 x manuel d'instructions

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SISSEL® GmbH · Bruchstrasse 48